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22

The F22 device

The F22 system consists of a series of aligners, thin removable orthodontic appliances, that are made-to-measure. The aligners allow the patient's teeth to be moved gradually from their original position to an optimal position.

Indications for use

The F22 system is indicated for the alignment of teeth during orthodontic treatments for malocclusions, and is available only with a dental prescription.

Contraindications

The F22 system is contraindicated in cases of periodontal pathologies, radicular resorption and pharmacological therapies or systemic conditions that may affect tooth movement.

Inserting your aligners

1. Carefully wash your hands with soap and water before touching the aligners.
2. Take hold of only one aligner at a time.
3. Rinse the new aligners thoroughly with cold water before use, after removing them from their packaging. All aligners are nevertheless cleaned, disinfected and packaged before consignment.
4. Check that the correct aligner has been taken. The upper aligner is for the upper dental arch, and the lower aligner is for the lower dental arch.
5. Either the upper or the lower aligner can be inserted first. To insert an aligner, push it delicately onto the incisors (front teeth). Then use the fingertips to apply a uniform pressure on the molars (back teeth) on the left and right, until the aligner clicks into position.
6. Do not bite on the aligners to position them correctly, because this could damage them.

Removing your aligners

1. Using the fingertips, start on one side of the molars and delicately remove the aligner, proceeding towards the other side.
2. To avoid damaging your aligners, remove them only if necessary (meals and oral hygiene).
3. Do not use pointed objects to remove your aligners.
4. Immediately rinse the aligners with abundant cold water and replace them in the holder provided in the initial kit.
5. Take care while removing your aligners, particularly if they have multiple attachments.
6. Do not excessively force, bend or twist the aligners while removing them.
7. If the aligners are particularly difficult to remove, consult your dentist.

Maintenance of F22 aligners

Aligners are a protected environment that limits the flow of saliva, impeding the normal cleaning process that this performs, as well as its buffering action and re-mineralizing properties.

In addition, when aligners are worn, the normal cleaning activities performed by the lips, cheeks and tongue are also interrupted, further promoting the accumulation of plaque beneath the aligners.

It is therefore essential to thoroughly clean the aligners.

Cleaning the aligners

1. Clean aligners every time before insertion using a brush with soft bristles and cold water (never hot) and a small amount of toothpaste.
2. Thoroughly rinse each aligner after cleaning.
3. Do not clean aligners using cleaning products for dental prostheses, or to soak them in mouthwash. These products can damage aligner surfaces, making them opaque and more visible.
4. It is advisable to use specific products to clean your F22 aligners at least once a week and whenever necessary. Any white deposit accumulating in or on the aligners is plaque that must be removed. Effervescent tablets can clean parts of aligners that are difficult to reach with a brush, such as for example the areas of the cusps.
5. If the aligners being used are excessively stained with colorants before the two weeks have elapsed, consult your dentist. Give your aligners to your dentist during the appointment fixed, and they can be cleaned in an ultrasonic cleaning bath. Your aligners will be ready at once for you to wear again.

Correct habits and oral hygiene

1. Always remove your aligners while eating or drinking. Aligners need not be re-moved to drink cold water. Coloured drinks (e.g. coffee, tea, red wine) should NOT be drunk, to avoid staining the aligners.
2. Unlike conventional brackets and wires, eating and drinking anything you want is always possible, because your aligners can be easily removed. It is therefore not necessary to limit the consumption of your favourite foods or drinks.
3. Always remove your aligners if you wish to smoke, to prevent yellow stains on them.
4. It is not advisable to chew gum while wearing your aligners, because it could stick to them and detach them.
5. Brush the teeth and pass dental floss between them after every meal or snack, before inserting your aligners again. If you are temporarily without your usual dental hygiene system or a toothbrush, rinse your mouth and clean the aligners with running cold water.
6. For all questions about oral hygiene techniques, consult your dentist.
7. Dental check-ups and cleaning should be performed periodically to guarantee the health of teeth and gums.

Conservation of F22 aligners

- The initial kit includes an aligner holder. Aligners should be replaced in this holder when not been worn, to reduce the risk of losing or damaging them.
- Immediately inform your dentist if the aligners currently worn are lost or broken. Your dentist may advise you to wear the previous pair of aligners until they can be replaced, or the aligners of the next phase, depending on how long the current aligners have been worn. As an alternative, your dentist may order a new pair of aligners to replace the lost ones, and these will be available in a couple of days.
- Always wear your current aligners at every appointment with your dentist, and also bring the previous aligners with you, each in its respective packaging bag. Your dentist will take the previous aligners and will give you the aligners for the next treatment phase. The aligners worn most recently must then be replaced in their packaging bag to take home with you.
- Immediately inform your dentist if an aligner attachment is lost or broken. Your dentist will probably repair the attachment, or may decide that it is no longer necessary for the subsequent treatment phase.

Caution

- In rare cases, some patients may be allergic to the plastic from which aligners are made.
- In rare cases, in patients affected by hereditary angioedema (HAE), rapid local swelling of subcutaneous tissues may occur, including the tissues of the larynx, because HAE may be activated by slight stimuli such as dental treatments.
- In this case, the treatment must be suspended, immediately consulting the dentist.
- Orthodontic devices or parts of them may be accidentally swallowed or sucked in, with harmful consequences.

Precautions

1. Aligners must not be exposed to hot water and to chemical materials.
2. Aligners must be kept out of reach of children and animals.
3. Always store aligners in a cool dry place.
4. Treatments for serious cases of open bites, overjet, mixed dentition and/or narrow jaws may require other procedures in addition to treatment with aligners.
5. Additional orthodontic treatments such as the use of fixed buttons, orthodontic elastics, auxiliary dental devices (e.g. temporary anchorage devices), removable or fixed devices and/or dental reconstruction procedures may be necessary for more complex treatment plans in which aligners alone would not be sufficient to achieve the desired results.
6. Oral surgery may be necessary to correct dental crowding or severe mandibular asymmetries which were already present before using the F22 product. Should oral surgery be required, the risks associated with anaesthesia and the correct healing of relevant tissues involved must be taken into consideration prior to treatment.
7. Already existing dental restorations (e.g. crowns or bridges) may move, making it necessary to re-cement or in some cases replace them.
8. Aligners cannot be used to move dental implants.
9. The patient's general medical conditions and the use of medicines may interfere with the orthodontic treatment.
10. Some tooth sensitivity may be experienced when passing on to the next aligner. Some people may complain about a slight temporary discomfort for some days at the beginning of each new treatment phase, when a new aligner is worn: this is a normal condition, usually described as a sensation of pressure. It is a sign that the aligners are working and are gradually moving the teeth towards the final result. The discomfort generally disappears within a couple of days.
11. Aligners may have a temporary effect on pronunciation and cause a slight lisp, even though any speech defects caused by using F22 tend to disappear within one or two weeks. The majority of patients have reported no effects on their pronunciation. Nevertheless, as with all conventional orthodontic devices, there is an initial adaptation period associated with the presence of a foreign body in the oral cavity.
12. Poor patient cooperation in not using the aligners for the prescribed number of hours per day and not following the doctor's advice, tooth eruption, teeth with atypical shapes and the failure to keep appointments could extend the treatment period and compromise the desired final result.
13. Gingivae, cheeks and lips may be scratched or irritated by the device.
14. To facilitate tooth movements and/or aligner retention, attachments may be temporarily fixed to one or more teeth, or divots may be created on aligners during treatment. The protuberances are located at points where the aligner adheres to the attachment positioned on teeth by the dentist. Attachments are small elements made of composite materials that the dentist fixes onto the teeth at points where they engage the aligner's protuberances. Divots are thin indentations situated on aligner surfaces which are made by the dentist using special forceps. Dentists use one or both of these techniques to obtain the desired tooth movements. Such attachments are removed after treatment.
15. Should an interproximal reduction be required in order to create the necessary spaces for teeth movements, the risks associated with the technique should be taken into consideration.

16. Periodontal problems, the formation of caries, gingival inflammation or permanent marks due to staining and tooth decalcification may affect patients who consume foods or drinks with a high sugar content and/or who do not clean their teeth frequently and thoroughly before wearing F22 aligners. Orthodontic devices per se do not cause caries or gingival inflammation, but their presence may give rise to a greater accumulation of bacterial plaque. It is therefore essential to scrupulously respect oral hygiene routines.
17. The bite may change during treatment and cause a temporary discomfort for the patient.
18. At the end of the treatment, the bite may require additional adjustments by the dentist. ("balanced occlusion").
19. The teeth may change position after the treatment has ended. It is advisable to use containment techniques. Teeth may tend to move after the treatment has ended. The constant use of containment aligners at the end of treatment will reduce this risk. However, there may be other causes beyond the orthodontist's control that can even cause major changes. Hence, the containment phase is just as important as the treatment phase, and must last as long as possible.
20. Teeth that are not at least partially covered by the aligner may be subject to super-eruptions.
21. The use of aligners may cause a temporary increase in salivation or xerostomia and some medicines may increase this tendency.
22. Periodontal problems (gingivae and supporting bone) may occur or worsen during the orthodontic treatment due to several factors, especially if one does not observe a proper oral hygiene. The dentist must therefore perform regular check-ups every 2–3 months to monitor the health of the periodontium, also ensuring that the patient maintains high standards of oral hygiene. If this regime is followed, the orthodontic treatment will not have any negative effects and it may even improve the overall oral situation, also due to the greater ease of cleaning correctly aligned teeth. If it is impossible to control the periodontal problems, the orthodontic treatment must be interrupted before completion.
23. The roots of the teeth may become shorter during the orthodontic treatment. This does not normally have any consequences, but very rarely it may constitute a problem, with a loss of pulp vitality and a significant decrease in the useful lifespan of the tooth or teeth affected by this shortening.
24. Very rarely, problems with the temporomandibular joint may also be encountered, with joint pains, headaches or ear problems. Such ailments may be experienced during the orthodontic treatment as well as at any other moment of one's life. The role of contact between teeth is unclear, since currently there is no scientific evidence that indicates its importance in the onset of this pathology.
25. In case of acute pain or severe discomfort, discontinue use of aligners and consult your doctor.
26. In rare cases, some patients may be allergic to the plastic from which aligners are made. In such cases, discontinue treatment and consult a doctor immediately.
27. Orthodontic devices may be wholly or partly accidentally swallowed or sucked in, with damaging effects.

Rules of use and patients' responsibilities

Sweden & Martina S.p.A. does not provide medical, dental, health or similar services, nor does it perform medical or dental practices or provide medical advice. Sweden & Martina S.p.A. exclusively manages the contact platform and manufactures the products according to the indications provided by the dentists. The implementation of any suggestion provided by Sweden & Martina S.p.A is left exclusively to the discretion of the dentist, who is the only party who may decide on how to treat the patient.

Any liability for therapeutic choices and treatment plans will be attributed solely to the dentist. Some products and/or treatment options incorporate predefined or pre-established treatment protocols, which are accepted by the dentist with the approval of the digital treatment simulation. Information on treatment protocols is available on the F22aligner.com website or when the order is sent. The order will become binding only after acceptance by Sweden & Martina S.p.A, which reserves all rights in this respect. Sweden & Martina S.p.A treatment options are intended solely for officially accredited dentists. Sweden & Martina S.p.A treatment options are available only for dentists regularly complying with payments and operating in conformity with the corporate policies and practices adopted by Sweden & Martina S.p.A. At the end of treatment, Sweden & Martina S.p.A reserves the right to ask dentists for a feedback on the treatment of their patients with the F22 system.

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Sweden & Martina Ltd - United Kingdom

info.uk@sweden-martina.com

Sweden & Martina Inc. - Distributor for U.S.

info.us@sweden-martina.com

F22 aligners are custom-made medical devices. Sweden & Martina is registered with the Italian Health Ministry as a manufacturer of custom-made medical devices. The Declaration of Conformity consigned to the client guarantees the conformity of the product. Contents are correct at the time of publication. Please contact Sweden & Martina SpA for information on any subsequent updates.

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A atuação de qualquer sugestão facultada por parte de Sweden & Martina S.p.A., é deixada unicamente à
A encomenda só será vinculativa após aceitação por parte da Sweden & Martina S.p.A. que se reserva o direito.